

USE AND MAINTENANCE MANUAL

PHYSIO ROLL



MODELS - DIMENSIONS

Diameter	Length	Colour	Code
30 cm	50 cm	Blue	1411071
40 cm	65 cm	Red	1411072
55 cm	90 cm	Yellow	1411073
70 cm	115 cm	Blue	1411074
85 cm	130 cm	Red	1411075

FISIOTECH SRL

Operating headquarters: Via Toniolo, 44/E-F-G - 31028 Vazzola (TV) - ITALY

Registered office: Via Valdoni, 19 - 31013 Cimavilla di Codognè (TV) - ITALY

Tel. +39 0438470342 - Fax +39 0438478762 - mail : info@fisiotech.com - pec : fisiotech@emailsec.it

VAT number IT 03765100262 - Rea TV 296956

Table of contents

INTRODUCTION	p. 3
MODELS – SAFETY	p. 4
WARRANTY – WORKING LOAD – INFLATION	p. 5
WARNINGS – CLEANING – ACCESSORIES	p. 6
EXERCISES	p. 7
MASSAGE – EXERCISES FOR CHILDREN	p. 8

INTRODUCTION

The purpose of this manual is to provide users and operators with the necessary information to use the Physio Roll properly, autonomously, and safely.

This manual contains information on the technical features, operation, maintenance, spare parts and security of the Physio Roll.

This manual must always be carried with the equipment and stored with care so that the readability is not compromised.

In case of transfer of use to a third party, this manual must be transferred as well, along with the other documents, as provided by EU Regulation 2017/745.

Intended use

This device is a specific support to the rehabilitation from neurologic and orthopedic illnesses, the treatment of muscular dystrophy, rehabilitation and pain relief gymnastics, strengthening and maintenance activities, Pilates, functional training and athletes. Class I Medical Device.

Manufacturer's details

LEDRAPLASTIC SPA

Distributor's details

FISIOTECH SRL

Operating headquarters: Via Toniolo, 44/E-F-G

31028 VAZZOLA (TV) - Italy

Tel. +39 0438 470342

Fax +39 0438 478762

info@fisiotech.com



WARNING! Thoroughly read the manual before use.

FISIOTECH SRL refuses any responsibility and is not liable in case any damage to people and/or goods caused by the improper use of the Physio Roll, by negligent or not-qualified staff or by non compliance with these instructions occurs.

MODELS

CODE	DESCRIPTION
1411071	Ø 30 cm L. 50 cm Blue
1411072	Ø 40 cm L. 65 cm Red
1411073	Ø 55 cm L. 90 cm Yellow
1411074	Ø 70 cm L. 115 cm Blue
1411075	Ø 85 cm L. 130 cm Red

SAFETY



IN CASE OF PUNCTURE, THE BALL BURSTS.



WARNING:

- Choking hazard. This product is not suitable for children below 36 months. It contains small parts that may be inhaled or swallowed.
- Only use for exercises with the weight of the body. Not suitable for exercises with dumbbells and other weights. Lifting weights while using the product may harm the user.
- An adult should supervise children when they are using the products.
- In case of puncture, do not repair. The repaired product loses the safety requirements.
- Do not expose the product to sources of heat or light for a long time, as it would alter its physical and colour features.


WARRANTY

Physio Rolls have a warranty that covers only the manufacturing defects for 12 months starting from the date of delivery. The warranty does not cover: wrong or improper use, use by non qualified personnel. The warranty is void if the device has been tempered with. Warranty repairs are carried out at our operational headquarters and prior authorization. The products to be repaired under warranty must be shipped to the manufacturer freight prepaid and with the original packaging. A copy of the invoice and of the transport document has to be sent along with a written request in order for the products to be repaired.

WORKING LOAD



INFLATION

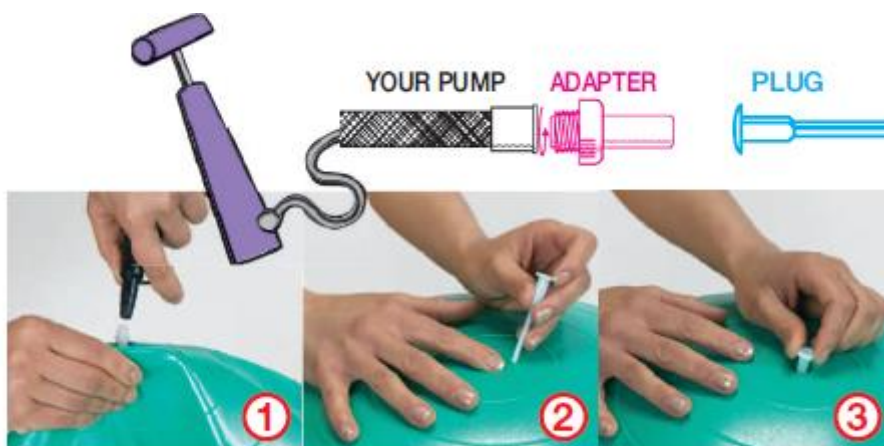

WARNINGS



Before every use:

- Verify the integrity of the product;
- Verify that you are far from sources of heat and sharpened objects (zip, rings, watches, etc).

CLEANING

Clean with water and soap.


Do not use brushes, abrasive sponges or chemical substances.

Do not use sterilisation systems.

OPTIONAL ACCESSORIES

CODE	DESCRIPTION
1411160	Needle pump
1411161	Manual pump ideal for medium- and big-sized balls

EXERCISES

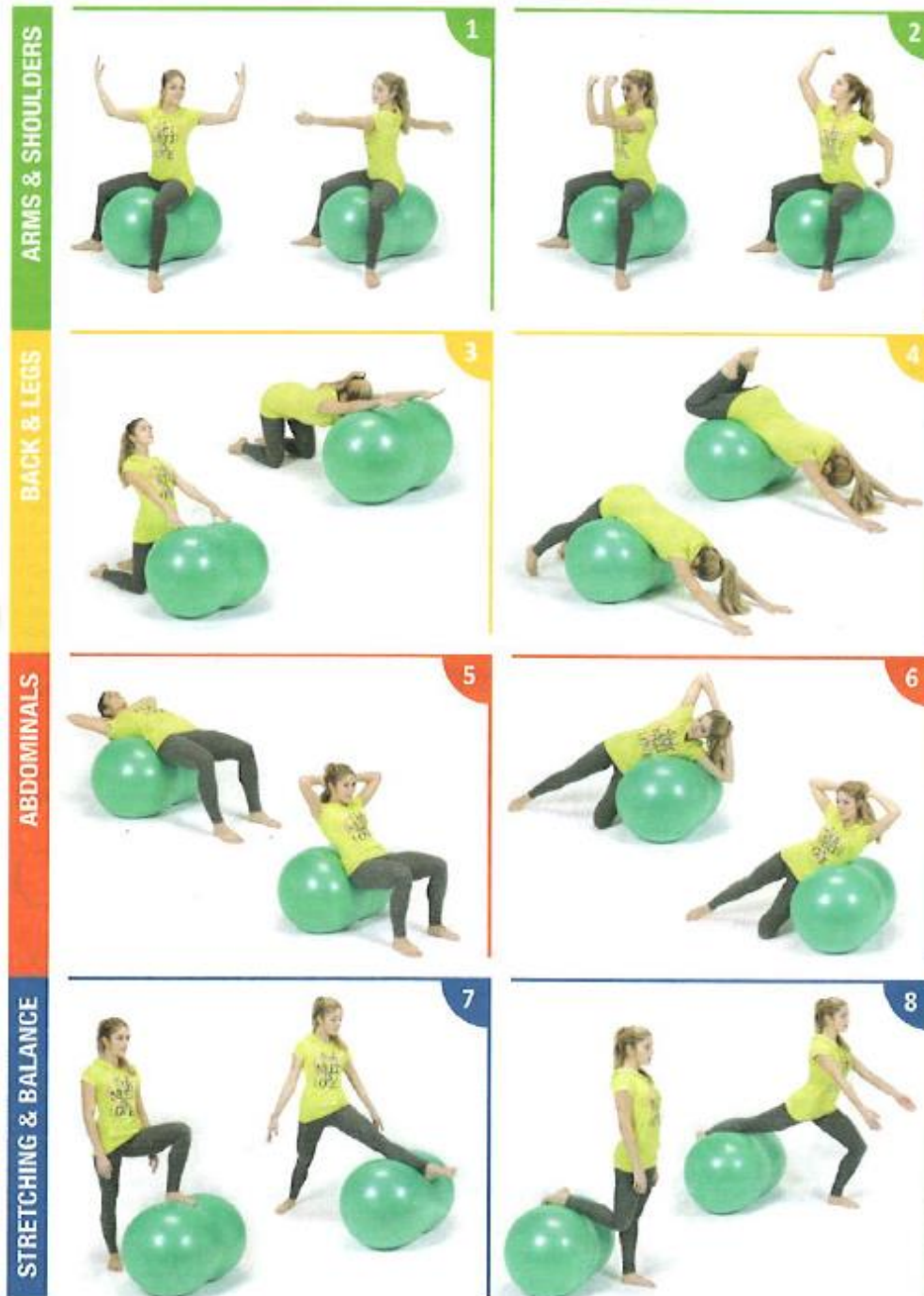

MASSAGE



EXERCISES WITH CHILDREN

